




# Universal Nasal Home Collection Kit Instructions

For use under Emergency Use Authorization

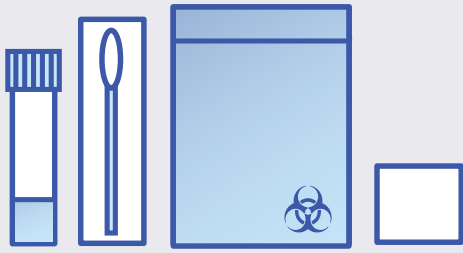

## Each kit bag contains:

One (1) transport vial containing media, one (1) swab,  
one (1) biosafety bag, and one (1) absorbent pad.

*Kit materials may differ in appearance based on what supplies are  
available at a given time.*

1



### Watch the Instructional Video

Visit [www.unh.edu/covidtest](http://www.unh.edu/covidtest) and watch  
the instructional video.

2



### Label

Fill out one barcode label with the date  
and time of sample collection. You  
received a sheet of personalized labels  
with your kits.

3



### Prepare

Affix one label on the vial. Cover the barcode  
that is already on the vial.

*\*Labels must be on the vial for sample to  
result\**


4



### Prior to Test

Wash hands with soap and  
water for 20 seconds and dry,  
or use hand sanitizer.


5

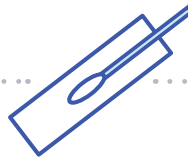

### Prep for Collection

Remove swab from packaging.  
Open from the side opposite the  
swab tip.


6

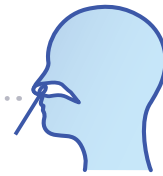

### Sampling

Rotate the swab in one  
nostril for 15 seconds  
(as shown in the video).


7

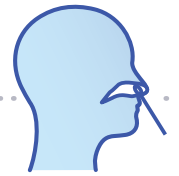

### Sampling

Rotate the swab in  
the other nostril for  
15 seconds.

7

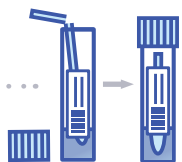

### Swab Securing

Place the swab (tip down) in the vial. Snap the  
swab handle by bending it firmly against the lip of  
the vial and dispose of the remaining piece. Screw  
the lid on the vial to seal securely. Make sure the  
top of the handle sits below the lip of the vial so  
the handle is not bent when the lid is sealed.  
NEVER fold the swab handle over to fit it in the vial.


8



### Sample Securing

Place the sealed tube in the  
biosafety bag along with the  
absorbent pad and seal the bag. (In  
some kits, the absorbent pad may  
already be inside the biosafety bag.)

9

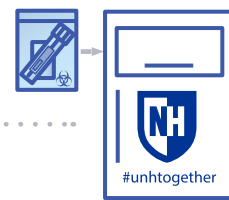

### Sample Drop-Off

Drop the sealed biosafety bag in the secured  
drop box at any collection point **within 2  
hours of sample collection**. Please use the  
drop boxes located on the campus where you  
received your kits (e.g., if you received your kits  
in Durham, please use the Durham drop boxes).



## Test kit drop box locations:

### Durham Campus

- > Dimond Library: Inside the first doors
- > Hamel Recreation Center: Lobby
- > Health & Wellness: Inside side entrance
- > Gregg Hall
- > West Edge Hut

### UNH Manchester Campus

- > Outside Front Entrance of 88 Commercial Street

### UNH School of Law Campus

- > Outside Main Lobby of 2 White Street

## Consent Form:

Everyone must have valid consent to test and receive results:



UNH Student/Employee Consent Form  
<https://covid-19.unh.edu/consent-for-testing>



CONTRACTOR Consent Form  
<https://covid-19.unh.edu/consent-for-testing>

## Frequently asked questions

### What happens if I test positive for COVID-19?

- > UNH Health & Wellness will contact you with further instructions. They will also contact you daily to monitor your symptoms, either by voice call, text or email.
- > Health & Wellness will help you connect with Academic Affairs to answer questions about classes
- > If you tested positive on a test outside of UNH upload to H&W using this form: <https://covid-19.unh.edu/upload-my-test-result>

### What happens if the test tube leaks when I flip it over to check the seal?

- > You need to complete a new testing sample.

### What if my sample spills, is rejected, or I've run out of test?

- > Test kits will be provided periodically throughout the semester, but you will not be given any extra. If your sample has been compromised or a kit is no longer usable, please contact Health & Wellness at (603)-862-9355.

### What is the absorbent pad for?

- > The absorbent pad is included to soak up spilled liquid in the event a sample leaks. This helps protect lab workers and prevents the contamination of other samples.

### What happens if I am experiencing COVID-19 related symptoms?

- > You should self-isolate immediately and call Health & Wellness at (603) 862-9355 to be evaluated.

## Contact information:

Visit UNH Health & Wellness website for more information at [www.unh.edu/health](http://www.unh.edu/health) or contact [covid@unh.edu](mailto:covid@unh.edu).

For issues with your testing portal, contact [unhcnidlab@unh.edu](mailto:unhcnidlab@unh.edu)

For all COVID related academic concerns, contact the Associate Dean of your college.



COVID Incident Reporting Form  
[tinyurl.com/UNHCovid](http://tinyurl.com/UNHCovid)



Upload your vaccine card  
[covid-19.unh.edu/UploadVaccine.html](http://covid-19.unh.edu/UploadVaccine.html)